

Cannulated Recon Cancellous System SURGICAL TECHNION



# THE ART of TRAUMA SURGERY

*The Art of Trauma Surgery* is a collaborative project between ITS. and Austrian artist Oskar Stocker that celebrates the skill, perseverance, and artistry of surgeons and engineers who work tirelessly to improve outcomes for trauma patients.

At ITS. we stand for long-term, trusting relationships with our customers, suppliers, and development partners. Through our devotion to innovation and development, we continuously seek to improve and optimize products and techniques in the field of traumatology.

We believe that the success of our mission lies in the combination of the technical expertise, compassion and dedication of surgeons and engineers to help patients regain their health and well-being. Join us in celebrating these remarkable individuals and *The Art of Trauma Surgery!* 

### About the Artist

The Austrian artist Oskar Stocker (b. 1956) lives and works in Graz, Austria. He has become known internationally through the exhibition Facing Nations, which consists of portraits of more than 120 people of various nationalities living in Graz; it was shown first in Graz itself, then in Vienna, and later culminated in 2010 with its display at the UN Headquarters in New York City.

In addition to the portraits of individual people, he devotes himself to the depiction of landscapes and objects, down to the smallest detail.



## Table of Contents

### I. Introduction

- 8 System Overview
- 9 Properties
- 10 Indications
- 10 Contraindications
- I0 Time of Operation

### 2. Surgical Technique

- I2 Temporary Fixation with K-wire
- 12 Identification of Screw Length
- 13 Countersink Drilling
- 13 Screw Placement
- 15 Postoperative Treatment
- I5 Explantation

### 3. Information

- 18 Technical Information
- 19 Type II Anodization
- 20 Ordering Information



## Introduction

## • System Overview

ITS. provides various cannulated screw types with different diameters and lengths covering a wide range of indications for fracture treatment in trauma surgery.

In addition to our existing cannulated screw range from 2.0mm up to 8.5mm, we now offer a 7.5mm fullythreaded cannulated screw. This screw is indicated primarily for percutaneous fixation of calcaneus fractures and secondarily for proximal tibia, distal femur fractures as well as arthrodesis and osteotomies of the upper and lower ankle joint.



### • Properties



## • Indications

- Primary: For percutaneous fixation of calcaneus fractures
- Secondary: For fixation of proximal tibia and distal femur fractures, as well as arthrodesis and osteotomies of the upper and lower ankle joint

## • Contraindications

- Diaphyseal fractures of long bones
- Advanced osteoporosis
- Obesity
- Lack of patient compliance

## • Time of Operation

- Primary: within the first hours after trauma
- Secondary: after swelling subsides, intermediate fixation with external fixation or extension

Intended purpose

The implant screw product group - S04 are used for the fixation of fractures, arthrodeses and osteotomies.

# Surgical Technique



## • Temporary Fixation with K-wire

After placing a stab incision, provisionally fix the fracture segments using the guide wire, Steel, D=2.5mm, L=230mm, TR (*35251-230*) and control its correct position under fluoroscopy.



## • Identification of Screw Length

Place screw depth gauge (59321) over guide wire and insert it through the soft tissue down to the bone. Then read off the required length measurement at the end of the calibrate guide wire.



## Ocountersink Drilling

Predrill the cortical bone using the contersink drill *(6360/-/50)* over the guide wire until the depth stop collar reaches the surface of the bone.

**NOTE**: After countersink drilling, the drill *(61552-180)* can be used to pre-drill the full screw length, if needed, thereby easing screw insertion in very hard bone.



### • Placement of the Screw

Insert the Cannulated Cancellous Recon Screw (3/75/-XX) over the guide wire using the AO silicone handle (530/6) and screwdriver shank T25 (54254-150 or 54255-150).



Finally, remove the guide wire and confirm correct screw position under fluoroscopy.



## • Postoperative Treatment

**IMPORTANT**: The postoperative treatment may vary depending on the patient's age, bone quality or type of fracture.

•	General:	Partial weight bearing at week 4-6 Full weight bearing at week 6-10	
•	Calcaneus:	Removable plaster cast to avoid pes equinus position Partial weight bearing (toe contact) Full weight bearing after week 10-16	15kg at week 0-6 30kg at week 6-10
•	Distal Femur:	Knee bandage in ligament fixation Partial weight bearing (toe contact)	15kg at week 0-6 30kg at week 6-10
•	Proximal Tibia:	Knee bandage in ligament fixation Partial weight bearing (toe contact)	I5kg at week 4-w8 30kg at week 6-I0
		Full weight bearing at week 10-14	

## • Explantation

Removal is possible, if desired by the patient. (Removal should be performed only after radiographic verification of the healed bone.)

The ITS. Type II anodization surface treatment reduces the risk of cold welding of titanium implants *(for more information, see page 27).* 



# Information



## • Technical Information



For detailed cleaning and sterilization instructions, please refer to package insert.

Not true to scale

## • Type II Anodization

Chemical process - anodization in a strong alkaline solution\*





#### Anodization Type II leads to the following benefits\*

- Oxygen and silicon absorbing conversion layer
- Decrease in protein adsorption
- Closing of micro pores and micro cracks
- Reduced risk of inflammation and allergy
- Hardened titanium surface
- Reduced tendency of cold welding of titanium implants
- Increased fatigue resistance of implants
- Improved wear and friction characteristics

\* White Paper: Ti6Al4V with Anodization Type II: Biological Behavior and Biomechanical Effects; Axel Baumann, Nils Zander

## • Ordering Information

### Cannulated Recon Cancellous Screw

### 

Description	Length (mm)	Article Number
Cannulated Recon Cancellous Screw, D=7.5mm	40	31751-40
Cannulated Recon Cancellous Screw, D=7.5mm	45	31751-45
Cannulated Recon Cancellous Screw, D=7.5mm	50	31751-50
Cannulated Recon Cancellous Screw, D=7.5mm	55	31751-55
Cannulated Recon Cancellous Screw, D=7.5mm	60	31751-60
Cannulated Recon Cancellous Screw, D=7.5mm	65	31751-65
Cannulated Recon Cancellous Screw, D=7.5mm	70	31751-70
Cannulated Recon Cancellous Screw, D=7.5mm	75	31751-75
Cannulated Recon Cancellous Screw, D=7.5mm	80	31751-80
Cannulated Recon Cancellous Screw, D=7.5mm	85	31751-85
Cannulated Recon Cancellous Screw, D=7.5mm	90	31751-90
Cannulated Recon Cancellous Screw, D=7.5mm	95	31751-95
Cannulated Recon Cancellous Screw, D=7.5mm	100	31751-100
Cannulated Recon Cancellous Screw, D=7.5mm	105	31751-105
Cannulated Recon Cancellous Screw, D=7.5mm	110	31751-110
Cannulated Recon Cancellous Screw, D=7.5mm	115	31751-115
Cannulated Recon Cancellous Screw, D=7.5mm	120	31751-120
Cannulated Recon Cancellous Screw, D=7.5mm	125	31751-125
Cannulated Recon Cancellous Screw, D=7.5mm	130	31751-130
Cannulated Recon Cancellous Screw, D=7.5mm	135	31751-135
Cannulated Recon Cancellous Screw, D=7.5mm	140	31751-140

All implants are available sterile-packed optionally. Add "-S" to the article number for sterile-packed implants (e.g. 37304-I2-S; 2103I-3-S). Delivery times, prices & minimum quantity vary from standard.

### Instruments

### Guide Wire

35251-230

-

Description	Article Number
Guide Wire, Steel, D=2.5mm, L=230mm, TR	35251-230

### Depth Gauge

									L		
20	30	40	50	60	70	80	90	100	110	12	2

59321

Description	Article Number
Depth Gauge 3.2mm, 6.5mm and 7.3mm Can. Screw	59321

### Spiral Drill

-----

61552-180

Description	Article Number
Spiral Drill, Cannulated D=5.5mm/D=2.6mm, L=180mm	61552-180

#### Cortical Counter Sink

63601-150

Description	Article Number
Cortical Counter Sink, L=I50mm, Cannulated, AO-Connector	63601-150

### Torque-Shank

	2	
54254-150	C) T25	
54255-150	T25	

Description	Article Number
Torque-Shank, T25xl50, T25xl50	54254-150
Torque-Shank, T25xl50, T25xl50, Cannulated	54255-150

### AO-Silicone Handle



Description	Article Number
AO Silicone Handle	53016

#### Disclaimer:

The intended users are limited to medical personnel with appropriate product training by the medical product consultants or knowledge of the surgical procedure to be applied. The medical staff must ensure that the use of I.T.S. GmbH medical devices is appropriate, taking into account the medical condition and medical history of the patient. Prior to product use, medical personnel must refer to complete information on product label and in IFU, including, but not limited to, indications, contraindications, warnings and preventative measures, and cleaning and sterilization instructions. Product availability is dependent on country registrations and clearances. For more information, please visit www.its-implant.com or contact us at office@its-implant.com. All information herein is the intellectual property of I.T.S. GmbH.

#### HEADQUARTER I.T.S. GmbH Autal 28, 830I Lassnitzhöhe, Austria Tel.: +43 (0) 316/ 211 21 0 office@its-implant.com www.its-implant.com

#### C € 0297 Order No. CRCS-OP-0125-EN Edition: January/2025

© I.T.S. GmbH Graz/Austria 2025 Subject to technical alterations, errors and misprints excepted.